

The Bliss Safety Harness is designed to both free the hands of the caregiver for bathing activities and to be a back-up safety device if the caregiver's attention is diverted away. The harness is fully adjustable and can be removed for cleaning or when not needed. The harness comes fully installed and only needs to be adjusted to fit the user.




HOW TO ADJUST THE SAFETY HARNESS:

1. Unbuckle the waist buckle by squeezing on the sides and pulling apart.
2. Seat the user in the tub and place the shoulder straps over the shoulders.
3. Keeping the buckle near the center of the user's waist, slide the two black plastic sliders down to the metal loop so that they catch when you gently pull on the buckle.
4. Position the comfort pads against the shoulders of the user.
5. Pull down on the opposite strap to secure the user against the back of the seat and then slide the other two black plastic sliders down to the metal loop. This sets the tension against the shoulders of the user.
6. Buckle the waist buckle and then gently lift up on the lip of the buckle as you pull the strap tight against the hips of the user.
7. If the left over length of strap is too great it can be trimmed to fit that user. Simply cut off the extra length of strap with scissors and then quickly sear the edge with a flame. The extra length may also be left to accommodate different size users.



Once the initial set up is done, simply buckle and unbuckle the user in and out of the tub. The strap is made of polyester webbing and is water resistant. Simply leave it in place and it will dry out on its own.

The strapping and the comfort pad both can be removed for cleaning and are washing machine safe. When replacing the harness, be sure to put the components back in the correct order. Refer to the diagram below for order of installation.

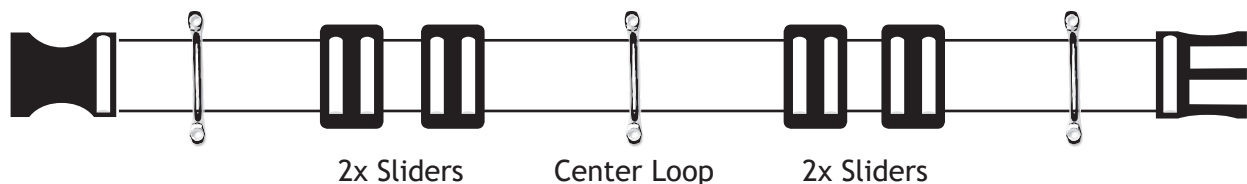

SAVE THESE INSTRUCTIONS