


## Bathing Pillow Cleaning Instructions

The bathing pillow should be cleaned at least once every 2 (two) weeks according to the following steps:

1. If you are using the suction cups (Fig. 1) to hold the bathing pillow in place, be sure to release them before removing the pillow for cleaning. Never pull on the pillow while it is attached to the tub.

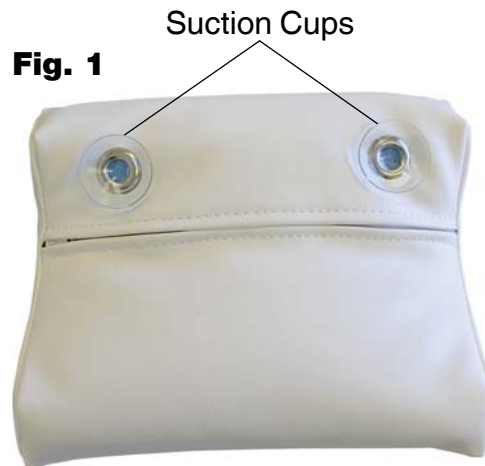

2. Prepare a cleaning solution by mixing 1 (one) tablespoon of liquid dish detergent and 2 (two) tablespoons of household bleach with 1 (one) gallon of water.
3. Remove the padding material (Fig. 2) within the pillow.



4. Clean the case exterior and interior surfaces with a sponge or soft cloth dipped in the cleaning solution, then rinse and let air dry.
5. Rinse the padding with water and set out separately to air dry.
6. Replace dry padding material into the case exactly as before and zip closed.

**SAVE THESE INSTRUCTIONS**