

See photos 1, 2 & 3 - Bathing seat cushion with suction cups (Caution: mold & mildew if cleaning instructions are ignored.) The tub seat cushion uses open-cell foam.

Cleaning Instructions (to be performed once every two (2) weeks.

1. If you are using the suction cups to hold the seat cushion in place, be sure to release them before removing the cushion for cleaning. Never pull on the cushion while it is attached to the tub.
2. Prepare a cleaning solution by mixing one (1) tbsp. of liquid dish detergent and two (2) tbsp. of household bleach with one (1) gallon of water.
3. Remove the padding material within the seat cushion, unfolding the three (3) layers. Note the order of the layers.
4. Clean the fabric case exterior and interior surfaces with a sponge or soft cloth dipped in the cleaning solution and then rinse in warm water and let air dry.
5. Rinse each padding material layer with water and then set out separately to air dry.
6. Replace dry cushion in case exactly as before and zip closed.

**1.****2.****3.**

Note: The shape of the seat cushion may vary depending on the tub model purchased.

**SAVE THESE INSTRUCTIONS**

© 2015 - 2017 Bliss Walk-in Bathtubs. All Rights Reserved.